


## Recommended Cameras for the "DELTA 20" using HEINE SLR Photo Adaptor

Manufacturer	Type	HEINE SLR Photo Adaptor
Canon	EOS 350 D* / 400 D*	K-000.34.185
	EOS 450 D	
	EOS 500 D	
	EOS 1000 D	
Nikon	D 40* / D 80*	K-000.34.186
	D 40 X	
	D 60 / D 90	
	D 200 / D 300 S	
	D 700	
	D 3000 / D 5000	
Olympus	E-450 / E-510	K-000.34.187
	E-520 / E-620	

\* no longer available

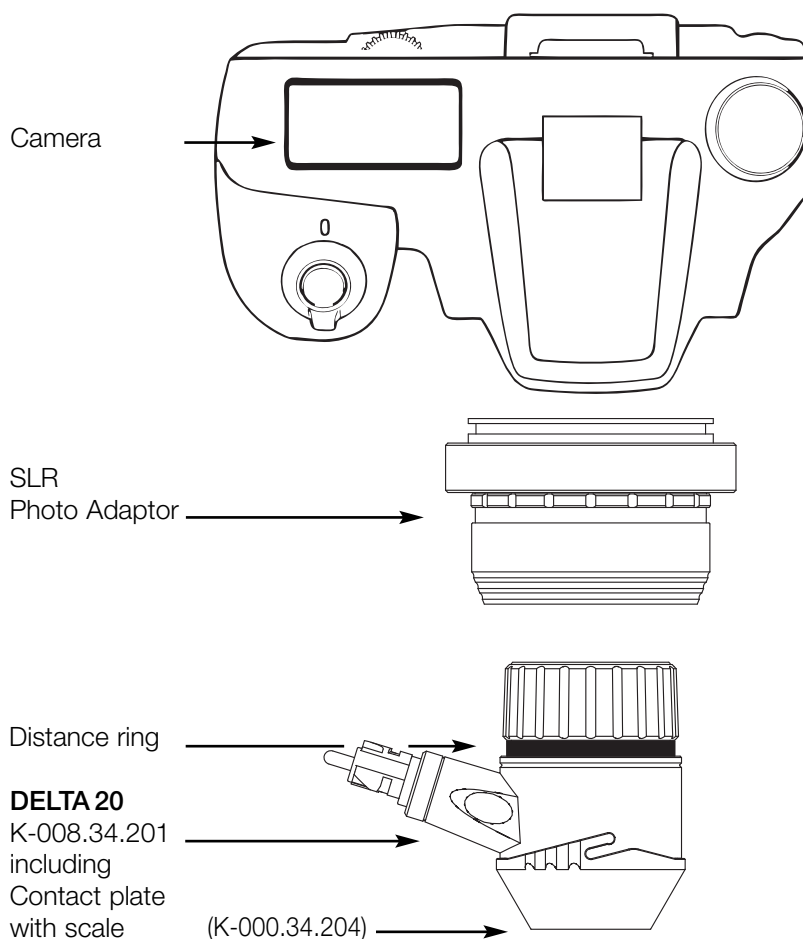

## Camera setup for SLR-Adaptor

### Nikon

- ISO: 400
- Program: M
- Exposure: 1/40s
- White balance: manual

### Canon

- ISO: 400
- Program: Tv
- Exposure: 1/40s
- White balance: manual

### Olympus

- ISO: 400
- Program: S
- Exposure: 1/40s
- White balance: manual